



#1
GLOBAL
BRAND¹



MENTOR[®]
MEMORYGEL[®]
SMOOTH
AND SILTEX[®]
MICROTEXTURE
BREAST IMPLANTS

Successfully Used
& Trusted for over

30
YEARS²



Not all Round Breast Implants are the Same

PROVEN

Performance

MENTOR® Breast Implants are backed by substantial clinical data demonstrating safety and effectiveness in both augmentation and reconstruction, including 10-year clinical trials.¹²

Design

Each profile in the MENTOR® MemoryGel® Breast Implant family contains proprietary and dynamic fill ratios developed from surgeon feedback.¹⁴

Peace of Mind

As one of the world's leading maker of high quality breast implants for over 30 years, our experience results in quality products that you can rely on and are backed by a comprehensive warranty.^{2,3}

Mentor® MemoryGel® Breast Implants in Primary Augmentation Patients

What does The Mentor® Level 2 Core Study Say at 10 years?



Made from a uniquely formulated, cohesive gel that acts more like a solid rather than a liquid¹³

Dynamic fill ratios for varying degrees of firmness⁶

MemoryGel® Breast Implant shells have 30% higher tensile strength versus another Breast Implant manufacturer shells¹⁷

Shell designed to minimize silicone gel bleed⁵

Automated manufacturing process for consistent quality¹⁵



Surfaces You Can Trust

MENTOR®
Smooth Breast Implants

Slower bacterial growth compared to Allergan®, Motiva®, Eurosilicone®, Polytech® POLYtxt®, Nagor® Nagotex®**⁹

Lowest adherence and growth for bacteria associated with capsular contracture and biofilm formation^{*9}

MENTOR® SILTEX®
Microtextured† Implants

Consistently demonstrate lower rates of BIA-ALCL compared with high-surface area textured implants⁸

¹ Head-to-head testing according to industry standard ASTM D412 test method for rubber properties in tension (v. 0901) between MemoryGel® (n=10) and Natrelle Inspira (n=10). Mentor Implant Shell Mechanical Properties between Allergan Natrelle and MemoryGel Xtra - August 2017

^{*} Based on in vitro testing as compared to Silimed Polyurethane, Eurosilicone™ Textured, Allergan® Biocell®, Nagor™ Nagotex®, Polytech POLYtxt®, Mentor® Siltex®, Motiva Implants® VelvetSurface™, Motiva Implants® SilkSurface®, Allergan® Smooth, Sientra® Smooth.

[†] According to ISO 14607.

Mentor® MemoryGel® Breast Implant Profiles, Cohesive I™

Smooth Round Moderate Plus



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer†
125	8.9	2.8	350-1251 BC	RSZ-1251
150	9.5	2.9	350-1501 BC	RSZ-1501
175	10.0	3.1	350-1751 BC	RSZ-1751
200	10.5	3.2	350-2001 BC	RSZ-2001
225	10.9	3.3	350-2251 BC	RSZ-2251
250	11.3	3.4	350-2501 BC	RSZ-2501
275	11.7	3.5	350-2751 BC	RSZ-2751
300	12.0	3.6	350-3001 BC	RSZ-3001
325	12.3	3.8	350-3251 BC	RSZ-3251
350	12.5	3.9	350-3501 BC	RSZ-3501
375	12.8	4.0	350-3751 BC	RSZ-3751
400	13.1	4.0	350-4001 BC	RSZ-4001
450	13.6	4.2	350-4501 BC	RSZ-4501
500	14.1	4.3	350-5001 BC	RSZ-5001
550	14.6	4.5	350-5501 BC	RSZ-5501
600	15.0	4.6	350-6001 BC	RSZ-6001
700	15.8	4.9	350-7001 BC	RSZ-7001
800	16.5	5.1	350-8001 BC	RSZ-8001

SILTEX® Microtextured Round Moderate Plus Cohesive I™



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer†
100	8.1	2.7	354-1001	
125	8.8	2.9	354-1251	RSZ-1251
150	9.4	3.0	354-1501	RSZ-1501
175	10.0	3.2	354-1751	RSZ-1751
200	10.5	3.3	354-2001	RSZ-2001
225	10.9	3.5	354-2251	RSZ-2251
250	11.3	3.6	354-2501	RSZ-2501
275	11.7	3.7	354-2751	RSZ-2751
300	12.0	3.7	354-3001	RSZ-3001
325	12.3	3.8	354-3251	RSZ-3251
350	12.6	3.8	354-3501	RSZ-3501
375	12.9	3.9	354-3751	RSZ-3751
400	13.2	4.0	354-4001	RSZ-4001
450	13.7	4.1	354-4501	RSZ-4501
500	14.1	4.2	354-5001	RSZ-5001
550	14.4	4.4	354-5501	RSZ-5501
600	14.7	4.5	354-6001	RSZ-6001
700	15.7	4.8	354-7001	RSZ-7001
800	16.6	5.0	354-8001	RSZ-8001

Smooth Round High



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer†
125	8.3	3.5	350-1254 BC	RSZ-1254
150	8.8	3.7	350-1504 BC	RSZ-1504
175	9.3	3.9	350-1754 BC	RSZ-1754
200	9.7	4.0	350-2004 BC	RSZ-2004
225	10.1	4.2	350-2254 BC	RSZ-2254
250	10.5	4.3	350-2504 BC	RSZ-2504
275	10.8	4.4	350-2754 BC	RSZ-2754
300	11.1	4.5	350-3004 BC	RSZ-3004
325	11.4	4.6	350-3254 BC	RSZ-3254
350	11.7	4.8	350-3504 BC	RSZ-3504
375	12.0	4.8	350-3754 BC	RSZ-3754
400	12.2	5.0	350-4004 BC	RSZ-4004
425	12.5	5.0	350-4254 BC	RSZ-4254
450	12.8	5.1	350-4504 BC	RSZ-4504
500	13.2	5.3	350-5004 BC	RSZ-5004
550	13.6	5.5	350-5504 BC	RSZ-5504
600	14.0	5.6	350-6004 BC	RSZ-6004
650	14.4	5.7	350-6504 BC	RSZ-6504
700	14.8	5.8	350-7004 BC	RSZ-7004
800	15.5	6.0	350-8004 BC	RSZ-8004

SILTEX® Microtextured Round High Cohesive I™



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer†
125	8.4	3.6	354-4125	RSZ-1254
150	8.9	3.8	354-4150	RSZ-1504
175	9.4	4.0	354-4175	RSZ-1754
200	9.9	4.1	354-4200	RSZ-2004
225	10.2	4.3	354-4225	RSZ-2254
250	10.5	4.5	354-4250	RSZ-2504
275	10.9	4.6	354-4275	RSZ-2754
300	11.1	4.7	354-4300	RSZ-3004
325	11.5	4.8	354-4325	RSZ-3254
350	11.7	4.9	354-4350	RSZ-3504
375	12.0	5.0	354-4375	RSZ-3754
400	12.3	5.1	354-4400	RSZ-4004
425	12.5	5.2	354-4425	RSZ-4254
450	12.7	5.2	354-4450	RSZ-4504
500	13.2	5.4	354-4500	RSZ-5004
550	13.5	5.6	354-4550	RSZ-5504
600	14.0	5.7	354-4600	RSZ-6004
650	14.3	5.8	354-4650	RSZ-6504
700	14.7	6.0	354-4700	RSZ-7004
800	15.4	6.3	354-4800	RSZ-8004

Smooth Round Ultra High



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer†
135	7.8	4.1	350-5135 BC	RSZ-5135
160	8.2	4.3	350-5160 BC	RSZ-5160
185	8.4	4.4	350-5185 BC	RSZ-5185
215	8.7	4.5	350-5215 BC	RSZ-5215
240	9.0	4.7	350-5240 BC	RSZ-5240
270	9.3	4.8	350-5270 BC	RSZ-5270
295	9.7	5.0	350-5295 BC	RSZ-5295
320	9.8	5.1	350-5320 BC	RSZ-5320
350	10.1	5.2	350-5350 BC	RSZ-5350
375	10.4	5.3	350-5375 BC	RSZ-5375
400	10.6	5.4	350-5400 BC	RSZ-5400
430	10.9	5.6	350-5430 BC	RSZ-5430
455	11.2	5.7	350-5455 BC	RSZ-5455
480	11.4	5.8	350-5480 BC	RSZ-5480
535	12.0	6.1	350-5535 BC	RSZ-5535
590	12.5	6.3	350-5590 BC	RSZ-5590

SILTEX® Microtextured Round Ultra High Cohesive I™



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer†
135	8.0	4.3	354-5135	RSZ-5135
160	8.4	4.5	354-5160	RSZ-5160
185	8.6	4.6	354-5185	RSZ-5185
215	8.9	4.7	354-5215	RSZ-5215
240	9.2	4.9	354-5240	RSZ-5240
270	9.5	5.0	354-5270	RSZ-5270
295	9.9	5.2	354-5295	RSZ-5295
320	10.0	5.3	354-5320	RSZ-5320
350	10.3	5.4	354-5350	RSZ-5350
375	10.5	5.5	354-5375	RSZ-5375
400	10.8	5.6	354-5400	RSZ-5400
430	11.1	5.8	354-5430	RSZ-5430
455	11.3	5.9	354-5455	RSZ-5455
480	11.6	6.0	354-5480	RSZ-5480
535	12.1	6.3	354-5535	RSZ-5535
590	12.6	6.5	354-5590	RSZ-5590

Mentor® MemoryGel® Breast Implant Profiles, Cohesive II™

SILTEX® Microtextured Round Breast Implants, Moderate Plus Profile, Cohesive II™



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer ¹
125	8.8	2.9	324-5125	RSZ-1251
150	9.4	3.0	324-5150	RSZ-1501
175	10.0	3.2	324-5175	RSZ-1751
200	10.5	3.3	324-5200	RSZ-2001
225	10.9	3.5	324-5225	RSZ-2251
250	11.3	3.6	324-5250	RSZ-2501
275	11.7	3.7	324-5275	RSZ-2751
300	12.0	3.7	324-5300	RSZ-3001
325	12.3	3.8	324-5325	RSZ-3251
350	12.6	3.8	324-5350	RSZ-3501
375	12.9	3.9	324-5375	RSZ-3751
400	13.2	4.0	324-5400	RSZ-4001
450	13.7	4.1	324-5450	RSZ-4501
500	14.1	4.2	324-5500	RSZ-5001
550	14.4	4.4	324-5550	RSZ-5501
600	14.7	4.5	324-5600	RSZ-6001
700	15.7	4.8	324-5700	RSZ-7001
800	16.6	5.0	324-5800	RSZ-8001

SILTEX® Microtextured Round Breast Implants, High Profile, Cohesive II™



Volume (cc)	Diameter (cm)	Projection (cm)	Catalog #	Resterilizable Gel Sizer ¹
125	8.4	3.6	324-4125	RSZ-1254
150	8.9	3.8	324-4150	RSZ-1504
175	9.4	4.0	324-4175	RSZ-1754
200	9.9	4.1	324-4200	RSZ-2004
225	10.2	4.3	324-4225	RSZ-2254
250	10.5	4.5	324-4250	RSZ-2504
275	10.9	4.6	324-4275	RSZ-2754
300	11.1	4.7	324-4300	RSZ-3004
325	11.5	4.8	324-4325	RSZ-3254
350	11.7	4.9	324-4350	RSZ-3504
375	12.0	5.0	324-4375	RSZ-3754
400	12.3	5.1	324-4400	RSZ-4004
425	12.5	5.2	324-4425	RSZ-4254
450	12.7	5.2	324-4450	RSZ-4504
500	13.2	5.4	324-4500	RSZ-5004
550	13.5	5.6	324-4550	RSZ-5504
600	14.0	5.7	324-4600	RSZ-6004
650	14.3	5.8	324-4650	RSZ-6504
700	14.7	6.0	324-4700	RSZ-7004
800	15.4	6.3	324-4800	RSZ-8004

A highly comprehensive protection plan...assured by Johnson & Johnson, a global leader in Healthcare with over 130 years of experience.^{10**}

7 MILLION+
women have
MENTOR® Silicone
Gel Breast Implants ¹¹



Providing patients with
proven safety, performance
and peace of mind.¹²

^{**} Based on warranty comparisons of breast implants. The third-party trademarks used herein are the trademarks of the respective owners.

REFERENCES:

1. Mentor Worldwide LLC. Mentor Worldwide Expander and Share Sales Data through Q2 2020. 2. Bondurant, S., Ernster, V., and Herdman, R. Safety of Silicone Implant. Washington, DC: National Academy Press. 1999. 3. MENTORPROMISE PROTECTION PLAN FOR MENTOR® BREAST IMPLANTS, Legal Document, 2021. 4. Mentor Worldwide LLC. MemoryGel® Core Gel Clinical Study Final Report, April 2013. 5. FDA: Summary of Safety and Effectiveness Data: Mentor MemoryGel® Silicone Gel-Filled Breast Implants. P030053. Recommended April 13, 2005. Notice of Approval November 17, 2006. <http://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpma/pma.cfm?id=p030053>, Accessed 23/11/2020. 6. Mentor_H572_Griffin_Moderate Plus Profile Gel_DOF_REF_20032801. Mentor_H572_Low_High Profile Gels Custom_DOF_REF_20010404. Mentor_H572_Weir_Design Input Requirements_DOF_REF_20110615. Mentor_TX820.130725.04...Determination_of_the_Gel...DOF_BENCH. 7. Mentor R&D Shell Tension Equivalency Testing - August 2017. Data on File. 8. Loch-Wilkinson A, Beath K, Knight RJW et al. Breast implant associated Anaplastic Large Cell Lymphoma in Australia and New Zealand - high surface area textured implants are associated with increased risk. Plast Reconstr Surg. 2016; Epub ahead of print. 9. Jones P, et al. The Functional Influence of Breast Implant Outer Shell Morphology on Bacterial Attachment and Growth. Plas Reconstr Surg. 142: 837, 2018. 10. Based on warranty comparisons of the following breast implants. Motiva® Warranty Program Terms and Conditions. Motiva, 2018. A Continued Commitment to Patients BIOCELL® Replacement Warranty. Allergan, 2019. Polytech. IMPLANTS OF EXCELLENCE PROGRAMME. Professional Service for Patients with POLYTECH Implants, 2019. GCA Comfort Guarantee. Nagor and Eurosilicone breast implants. Website: shorturl.at/aaGIR. Access date: 5/13/2020. About Johnson & Johnson. Website: <https://www.jnj.com/about-jnj>. Access date: 16/06/2020. 11. Mentor Worldwide LLC. Mentor Worldwide Historical Implant Data of Mentor MemoryGel® and Saline Units Sold (exclude Prouse) sold since 1985 -2020, January 2020. 12. MemoryShape Post- Approval Cohort Study (formerly Contour Profile Gel Core Study) Final Clinical Study Report. Mentor Worldwide, LLC; 02 June 2015. MemoryGel Core Gel Clinical Study Final Report. Mentor Worldwide, LLC; April 2013. Mentor MemoryShape Post-Approval Continued Access Study (formerly Contour Profile Gel Continued Access Study), Final Report. October 2014. Mentor MemoryGel Breast Implant Large Post Approval Study Re-Op Phase Annual Report. 17 June 2016. Adjunct Study Final Report for Mentor's MemoryGel Silicone Gel-filled Breast Implants. 02 November 2012. Mentor MemoryShape CPG Styles Study: A Study of the Safety of the Contour Profile Gel Breast Implants in Subjects who are Undergoing Primary Breast Augmentation, Primary Breast Reconstruction, or Revision, Final Clinical Study Report. 20 October 2015. 13. MENTOR Worldwide LLC. PROC400623 MENTOR Worldwide LLC. H572.021114.06, H572.010308.02, H572.010514.01 Email J. Lambert VP NuSil Technology. Dated 11.13.2018 14. Mentor_H572_Griffin_Moderate Plus Profile Gel_DOF_REF_20032801 Pg. 2. Mentor_H572_Low_High Profile Gels Custom_DOF_REF_20010404. Pg. 3. Mentor_H572_Weir_Design Input Requirements_DOF_REF_20110615. Pg. 7. Mentor_TX820.130725.04...Determination_of_the_Gel...DOF_BENCH. Pg. 3. 15. Mentor Worldwide LLC. TX820.130420.01.

IMPORTANT SAFETY INFORMATION

MENTOR® MemoryGel® Breast Implants are indicated for breast augmentation, in women who are at least 18 years old, or for breast reconstruction. Breast implant surgery should not be performed in those women with active infection anywhere in their body, those with existing cancer or pre-cancer of their breast(s), those who have not received adequate treatment for these conditions or those who are pregnant or nursing. There are risks associated with breast implant surgery. Breast implants are not lifetime devices and breast implantation are not necessarily a one-time surgery. The most common complications with MENTOR® MemoryGel® Breast Implants include re-operation, implant removal, capsular contracture, asymmetry, and breast pain. A lower risk of complication is implant rupture, which is most often silent. The health consequences of a ruptured silicone gel-filled breast implant have not been fully established. Screenings such as mammography, MRI, or ultrasound are recommended after initial implant surgery to assist in detecting implant rupture. Breast implants are also associated with the risk of breast implant-associated anaplastic large cell lymphoma (BIA-ALCL), an uncommon type of lymphoma. An individual's risk of developing BIA-ALCL with MENTOR® Breast Implants is low based on the incidence of worldwide cases.

Your patient needs to be informed and understand the risks and benefits of breast implants, and she should be provided with an opportunity to consult with you prior to deciding on surgery. For detailed indications, contraindications, warnings and precautions associated with the use of all MENTOR® Implantable Devices, please refer to the Product Insert Data Sheet provided with each product or review the Important Safety Information provided at www.mentorwllc.eu.

© Mentor® EMEA LLC 2021. Intended for use by or under the direction of a physician use, it is important to read the Instructions for Use and to understand the contraindications, warnings, and precautions. All other companies' brand names mentioned herein are the trademarks of their respective owners. 157754-201102. This publication is not intended for distribution outside of the EMEA region.